<b>Liebermann Reagenz</b> <b>Substance Tester - Drug Tester</b>
With this test you can in the simplest way <b>93 different liquids and solids</b> <b>Substances are tested</b>
<b>application</b> 1. The substance to be analyzed on a suitable one Place a base (glass or ceramic plate, etc.). <b>do not use any organic material such as wood, paper or cloth</b> <b>Organic material is attacked and decomposed</b> 2. Put one drop of the "Froehde" reagent on the sample. A color reaction occurs after approx. 5 - 10 seconds If there is one or more of the listed in the table Ingredients are in the sample. <b>see enclosed color table</b>
<b><u>Security delusions</u></b> When using the reagent: Protective gloves / protective clothing / eye protection Wear face protection - contains corrosive acids! Causes skin burns and eye damage If exposed or unwell: Immediately call a POISON CENTER or physician In case of contact with the eyes: Rinse gently with water for a few minutes
<b><u>Note :</u></b> The reagent is absolutely legal it does not contain any psychoactive or illegal substances The test kit is not suitable for scientific analyzes The test kit has a shelf life of approx. 6 months. (Storage cool and dark)

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